**softcaresilicone.org**

**Softcare Silicone UK Ltd**

## **Softleaves Triangular X100 Light-Weight Silicone Breast Forms**

Price per Unit (piece): £29.99

# **Softleaves®** *Triangular Light-Weight X100*

*High Quality Silicone Breast Forms*

***This sale is for the Softleaves® Triangular X100 LightWeight full Silicone Breast Form***

**For accessories especially designed for use with these forms( adhesive-tapes. ,shoulder stapes ,covers) please visit our softleaves shop**

***RRP for this item is 280 GBP for the pair in 2013, limited quantity available for the above very low price ( the above price is our clearance price )***

Features & Benefits

***Softleaves Triangular X100 Light Weight Silicone Breast Forms are high quality Full Silicone Breasts***

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**Breast Size:**

*This sale is for a pair ( two pieces) or one piece only ( depending on what you select from the drop box at the top of this page )*

**The approximate weigh of each side is 550 g.**

**Dimensions :**

**Width (side to side) : 23.5 cm or 9.1 Inch**

**Length (top to bottom) : 18 cm or 7.1 Inch**

**Thickness at the thickest point : 8.5 cm or 3.9**

**They will take you from a flat chest up to : 42E , 44DD , 46D , 48C**

**example : if your band size is 42 , then these breast forms will make you an E cup and you will need to use with them a bra size 42E**

**however, if your band size is 44 , they will make you a DD , and if the band size is 46 then will make you a D , and if the band size is 48 they will make you C .**

**What will you receive?**

**1- A pair or one peice only ( depending on what you select from the drop box) of Softleaves® Triangular X100 Light weight Silicone breast forms.**

**2- specially designed, very high quality box to protect the breast forms after use.**

**Please note that the item will be sent in discreet packaging .**

*Instruction for use:*

*Place the silicone breasts in a suitable bra, then wear the bra, the breasts will mould to your chest curvature.*

*Use Softeaves double sided adhesive tape(not included in this sale and found in our softleaves shop ) to attach these breast forms to the skin.( make sure you place sufficient tapes to cover the breast form to avoid sudden release and damage to the skin and forms- always using a bra to support the forms is recommended)*

*Use Softleaves Covers with these breast forms (not Included with this sale and found in our softleaves shop ) if you don't want them in direct contact with the skin.( you can also use the fabric breast pads that are included in this sale*

*These breast forms have their own nipples, however, if you like to make them more realistic, softleaves silicone nipples ( not included with this sale) can be used, simply moister the back of the nipples and attaching them to the breast form. If you need the nipples to attach firmly, please use, softleaves adhesive nipple pads (available from our online shop).*

**Taking Care of your Silicone Softleaves Triangular X100 Breasts :**

- *After use, keep the breasts in their box.*
- *Hand wash with warm soapy water*
- .- *Allow to dry naturally or use a hair dryer on a low temperature setting*
- .- *Keep away from excessive heat.*

*Please also visit our Softleaves shop for a full range of high quality silicone breast care products and accessories specially designed for this item (double side adhesive tapes , silicone nipple enhancers, silicone bra ,and covers) .*

*please kindly not that our accessories are specially designed for our Pinkwaves and Softleaves breast enhancers and might not work for other kinds of breast enhancers available on the market, as our accessories and enhancers are made from very high quality materials and are especially designed to work perfectly when used with each other, for example, our softleaves double-sided adhesive tape will peel of the silicone breast enhancers without effecting our enhancers , while if used with other low quality products , it might damage them.*

# Softleaves® *Triangular X100 Light*

*High Quality Silicone Breast Forms*

*Softleaves is a UK registered brand for For Softcare Silicone UK Limited*

[Vendor Information](#)