

softcaresilicone.org

Softcare Silicone UK Ltd

Softleaves V100 Silicone Breast Forms

Price per Unit (piece): £39.99



Softleaves® V100

High Quality Silicone Breast Form

This offer is for Softleaves® V100 Silicone Breast Forms . They come as one peice only (if you want a pair , please order the left side , than order agin the right side).

Please kindly select from the drop box at the top of this page the breast form size , color , nipple colour and the set (left or right side) .

For accessories especially designed for use with these forms (Skin adhesive tapes. ,shoulder straps ,Breast form covers..) please kindly visit our Softleaves accessories shop .

RRP for this item is 70-240 GBP depending on the size , limited quantity available for the above low price

Softleaves V100 Silicone Breast Forms are Very High Quality Full Silicone Breasts

Features & Benefits :

- ***Designed as enhancers for women with flat chest (very small or no breasts).***
- ***Natural shape to match the curves of the chest cage giving perfect fit and comfort; therefore providing the feel and look of having large breasts.***

- *Asymmetrical , therefore, each side is designed to be used on the left or right side of the chest giving the most natural look possible .*
- *Undetectable, as they bounce, shape, and feel natural due to the ultra-soft silicone material which is encased in a thin polyurethane layer, providing the same homogeneous structure as the natural breast.*
- *Designed to be used with almost any bra .*
- *Easy placement, durable, water proof and warm up to the body temperature.*
- *Double side skin adhesive tapes (Such as the Softleaves double-sided skin adhesive tape found in the Softleaves accessories section) and most adhesive attachments can be used with these breasts to make them self adhesive .*
- *The breast forms have their own nipple impressions as can be seen in the photos , you can also select the nipple colour (the pink nipples are very realistic as can be see in the photos, however, the flesh nipples are just a nipple impression on the breast forms that have the same colour of the breast form as can be seen in the photos) .*
- *Two extra breast pads will be provided to be used, if you do not want the breast in direct contact with your skin.*

Breast Size :

Softleaves Breast Form Conversion Table

To find the Softleaves breast form size that is suitable for you , please kindly look for your bra size on the conversion table below. You can find the corresponding breast form size on the left column of the table.

Example: If your bra size is 42B you will require a size 8 breast form

The horizontal row shows the cup size (bust size) , and the vertical column shows the band size .

Please Kindly note that this table should be used as a guide only

	AA	A	B	C	D	DD	E
30	0	1	2	3	4	5	6
32	1	2	3	4	5	6	7
34	2	3	4	5	6	7	8
36	3	4	5	6	7	8	9
38	4	5	6	7	8	9	10
40	5	6	7	8	9	10	11
42	6	7	8	9	10	11	12
44	7	8	9	10	11	12	13
46	8	9	10	11	12	13	14
48	9	10	11	12	13	14	15
50	10	11	12	13	14	15	16
52	11	12	13	14	15	16	17

Dimensions and weight :

Please kindly note that this table should be used as a guide only and shows only the sizes that are currently available in stock .

Size	Breast Size	Weight each Piece	Dimensions (cm)
4	L	300 g	19 x 15 x 5
6	XXL	400 g	19.5 x 15.5 x 5.5
7	XXXL	500 g	20 x 16 x 6
8	4XL	600 g	20.5 x 16.5 x 6.5

The above table is to be used as a guideline only, item can be returned or exchanged with any size for no extra post costs or for a full refund within 30 days and in any condition .(this does not effect your statutory rights)

What will you receive?

1- **One piece of Softleaves® V100 Silicone Breast Form.**

please select the size ,colour , and the side (Left or Right) of the breast forms from the drop box at the top of this page.

2- **A pair of fabric breast pads.**

3- **A high quality black box to protect the breast forms after use .**

Please note that the item will be sent in discreet packaging .

Instruction for use:

Place the silicone breasts in a suitable bra, then wear the bra, the breasts will mold to your chest curvature.

Use Softeaves double sided adhesive tape (found in Softleaves shop) to attach these breast forms to the skin.(make sure you place sufficient tapes to cover the breast form to avoid sudden release and damage to the skin and forms- always using a bra to support the forms is recommended).

Taking Care of your Silicone Softleaves V100 Breasts :

- After use, keep the breasts in their box.
- Hand wash with warm soapy water
- .- Allow to dry naturally or use a hair dryer on a low temperature setting
- .- Keep away from excessive heat.

Please also visit our Softleaves shop for a full range of high quality silicone breast care products and accessories specially designed for this item (double side adhesive tapes , silicone nipple enhancers, silicone bra ,and covers) .

please kindly not that our accessories are specially designed for our Pinkwaves and Softleaves breast enhancers and might not work for other kinds of breast enhancers available on the market, as our accessories and enhancers are made from very high quality materials and are especially designed to work perfectly when used with each other, for example, our softleaves double-sided adhesive tape will peel of the silicone breast enhancers without effecting our enhancers , while if used with other low quality products , it might damage them.

RRP : 70-240 BP (this item is on offer for the above low price for a limited period of time)

<!--[if !supportLineBreakNewLine]-->

<!--[endif]-->

Softleaves[®] V100

High Quality Silicone Breast Forms

Softleaves is a UK registered brand for Softcare Silicone UK Limited

[Vendor Information](#)